Filter Concentrators, CytoFuge® 2

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FF01/FFR1/FF01-B

Cautions

 *Universal Precautions should be followed on all specimens, regardless of whether a specimen is known to contain an infectious agent.*

Intended Use

IVD For *in vitro* diagnostic use.

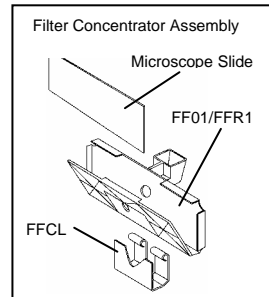
To concentrate cells onto a microscope slide used with the CytoFuge 2 cytocentrifuge.

Specimen Requirement

Sample Concentration	Sample Instruction
500-1500 cells/µL	Use 100-400µL
< 500 cells/µL	Pre-concentrate
> 1500 cells/µL	Dilute

Materials Required

- Filter Concentrators with filter attached (FF01) or filter separate (FFR1/FF01-B)
- Microscope Slide (with cell adhesion promoter, Superfrost Plus or equivalent)
- Stainless Steel Clips (FFCL)

Storage
5 °C  35° C

Procedure

Assembly of Filter Concentrator

- Unhinge the Filter Concentrator
- FFR1/FF01-B - Align Filter card hole with Concentrator hole.
- Place the Microscope Slide on the backside of the Filter Concentrator, frosted side up.
- Gently compress the assembly with one hand, and slide the Steel Clip over bottom of the concentrator.

Installation and Operation

- Install fully assembled Filter Concentrators (funnel side up) into the rotor opposite each other; if only one sample needs to be processed assure a "balance" assembly is seated opposite the sample.
- The Filter Concentrators should tilt towards the center of the rotor.
- Load sample after installing Filter Concentrators to minimize premature settling of cells.
- Screw lid on rotor with minimum force and close centrifuge cover securely.
- Press start button after choosing appropriate cycle.
- Consult the CytoFuge 2 Operator's Manual for complete instructions on the operation of the centrifuge.

Disassembly of Filter Concentrator
When the timed cycle is complete, the rotor will stop and the lid will release. At this time the latch can be squeezed to open the cover.
ALL SLIDES:

- Carefully remove the Filter Concentrator from the rotor (tilting funnel downward).
 - Remove the Steel Clip.
 - Unhinge the Filter Concentrator.
 - Carefully remove the microscope slide from the assembly.
 - Stain/treat the slide as desired.
- UNFIXED CYTOLOGY SLIDES ONLY:
- Inspect funnel area. A small amount of residual fluid should be present on the slide. (If the chamber is completely dry, decrease run duration and/or reduce speed on subsequent runs.)
 - Hold the assembly, funnel side up.
 - Remove the Clip.
 - Unhinge the Filter Concentrator and remove slide.
 - Allow residual fluid to flood the cells on the slide.
 - Remove the slide and stain/treat as desired.

General Cleaning and Reuse

- FFR1/FF01-B can be disinfected with 10% bleach (Can be reused 10 times)
- Filter Cards (FF01) (Single Use Only)

References

- "Protection of Laboratory Workers from Occupationally Acquired Infections; Approved Guideline – Second Edition." NCCLS document M29-A2 [ISBN 1-56238-453-8]. NCCLS, 940 West Valley Rd., Suite 1400, Wayne, PA 19087-1898 USA, 2001.

StatSpin®

StatSpin and CytoFuge are both registered trademarks of StatSpin, Inc., a Subsidiary of IRIS International, Inc.

Norwood, MA 02062 USA
Phone: 800-782-8774
or 781-551-0100
Fax: 781-551-0036
www.statspin.com

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mdi Europa GmbH